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Clean Active Aqua Plus one-phase solution for all types of soft contact lenses disinfects, preserves, rinses, cleans, moisturizes, removes daily muco-protein deposits
PROPERTIES: The one-phase solution CLEAN ACTIVE AQUA PLUS, thanks to the synergy between active ingredients and innovative components, has a strong disinfecting, preserving, and storing action. It is an isotonic and isohydric solution with respect to ocular cells, making it biologically tolerated even by very sensitive eyes. Its proven cleansing and proteo-mucolytic power ensures that the lens remains well- hydrated both immediately after application and during daily wear, ensuring better comfort. **INDICATIONS:** Indicated for storing, disinfecting, rinsing, cleaning, moisturizing, and removing daily muco-protein deposits from all types of soft contact lenses.

INSTRUCTIONS FOR USE:

1. Method 5-7 minutes

- Wash your hands and dry them thoroughly with a clean towel. • To avoid confusion, always start the treatment with the same lens. • Remove the lens from your eye and place it in the palm of your hand, adding enough CLEAN ACTIVE AQUA PLUS to completely wet the lens. • Gently rub both surfaces of the lens with a "forward and backward" motion for at least 10-15 seconds, using the finger of the other hand. • Rinse the lens thoroughly and abundantly with CLEAN ACTIVE AQUA PLUS to remove any traces of deposits. • Place the lens in the appropriate container. • Repeat for the other lens. • Fill the container with CLEAN ACTIVE AQUA PLUS up to the dosage line. • Close the container tightly and leave the lenses submerged for at least 5-7 minutes. • Before applying to your eyes, rinse the lenses with CLEAN ACTIVE AQUA PLUS.

2. "DON'T TOUCH" Method

- Wash your hands and dry them thoroughly with a clean towel. • To avoid confusion, always start the treatment with the same lens. • After use, remove the lens from the eye and immediately place it in the appropriate container filled with CLEAN ACTIVE AQUA PLUS up to the dosage line. • Repeat for the other lens. • Close the container tightly and leave the lenses submerged in the solution for at least 2-3 hours or overnight. • Before applying, rinse the lenses with CLEAN ACTIVE AQUA PLUS.

PRECAUTIONS:

CLEAN ACTIVE AQUA PLUS, once opened, should be used within three months. Any excess should be discarded. Do not use if the seal is broken, as sterility cannot be guaranteed. To avoid contamination, never touch the dropper with any surface. Close the bottle tightly after use. Store the bottle away from heat sources and at a temperature below 30°C. Do not use after the expiration date.

WARNINGS:

For lenses that have been stored in CLEAN ACTIVE AQUA PLUS solution for more than 3-4 weeks, before use: discard the solution from the container and perform disinfection at least according to Method 1 (5-7 minutes). Do not use in case of hypersensitivity to any ingredient. Keep out of reach of children. Always use fresh solution, discarding any remaining solution in the lens container. Do not replace with a different disinfecting product. Consult a professional before using any ophthalmic medication while wearing contact lenses. Never use tap water for soft contact lenses.

COMPLICATIONS: If discomfort or persistent irritation occurs, discontinue use and consult a specialist.

CONTRAINDICATIONS: There are no known contraindications, except in cases of allergy or hypersensitivity to any of the ingredients.

COMPOSITION: Polyhexamethylene biguanide: 0.0001%; Sodium EDTA: 0.1%; Anionic-cationic surfactants: 1.001%. Excipients and water up to 100%.

Follow the instructions on the label carefully and keep this leaflet.

Product shelf life: 3 years, if the packaging is intact and stored properly.

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